

JERRY HARVEY AUDIO IMPRESSION TIP SHEET FOR AUDIOLOGISTS



1 JH AUDIO manufactures a full shell earpiece. What we need from our customers is a full shell impression, including the crus of helix, tragus, and anti-tragus.

2 The impressions must be made of high-viscosity silicone impression material.
BLUE IMPRESSION MATERIAL NO LONGER ACCEPTED.

3 We prefer and recommend that the ear dams are made of foam.

4 It is required that we have an open mouth impression taken to the second bend of the ear canal. To ensure there is no jaw movement while the material is curing, a bite block must be used prior to the injection and throughout the entire process. To accomplish this:

A.) Customer's mouth should be wide open when the impression material is injected into the ear.

B.) Have the customer keep their mouth open until the impression has completely cured.

5 Want to know if your impressions are acceptable before you mail them to us? Text us at **321-320-5189**. Send your name and a few photos of your impressions to get initial approval for them. Final impression approval occurs upon arrival.

6 Send impressions to the address below. If you have any questions or comments, please contact us:



Jerry Harvey Audio
Attn: Lab
111 West Jefferson St, Suite 300
Orlando, FL 32801

Phone: 866-485-9111
Fax: 407-814-0018
www.jhaudio.com

Proceed to NEXT PAGE for visual guide.



**Impression of
1st & 2nd Bend**

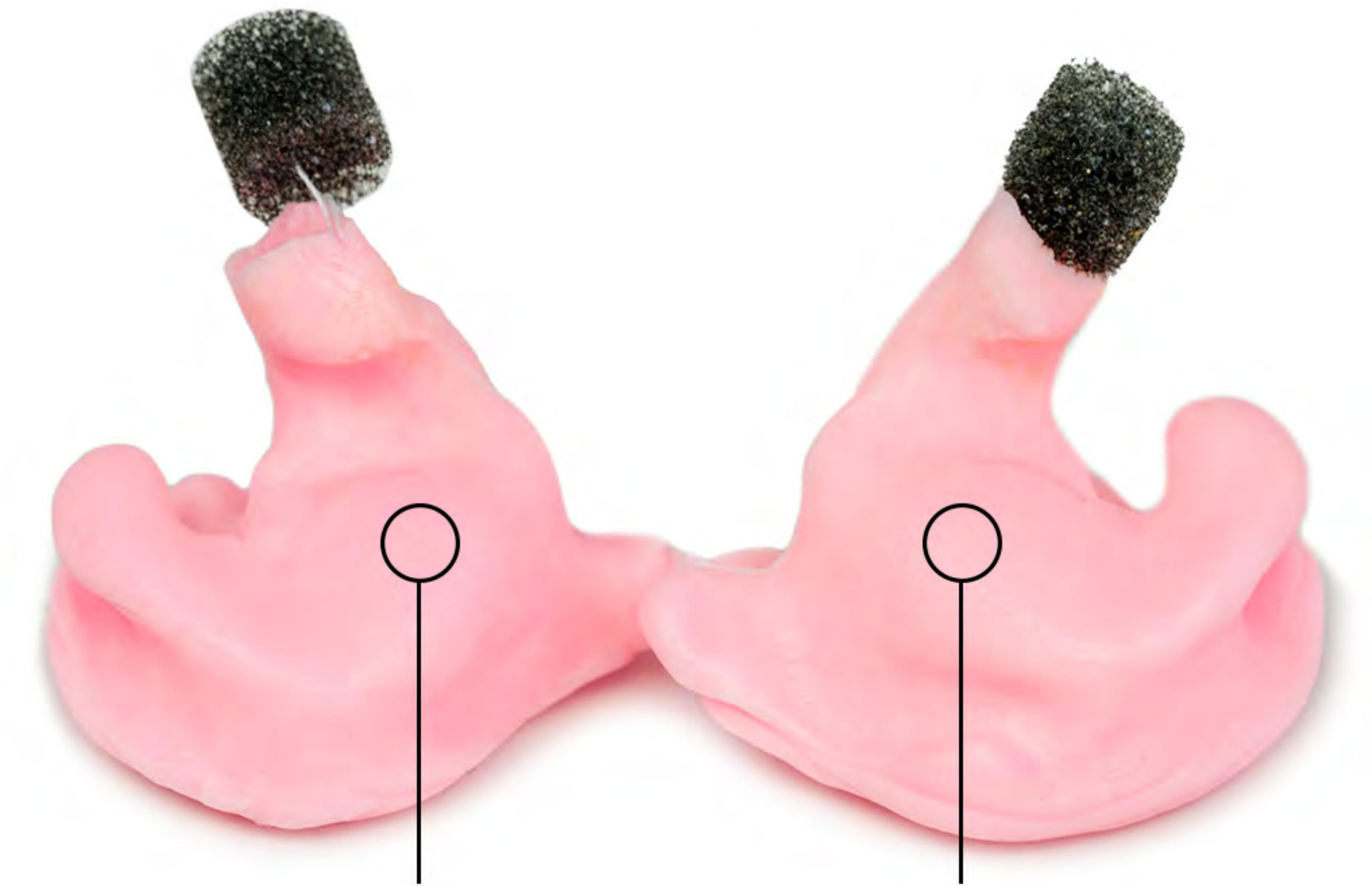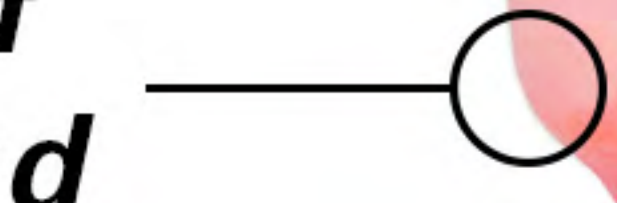

Concha Bowl Fully Formed



Defined Helix



**Extra Material Outside
of the ear, above Tragus.**